

24. Cummins, B., et. al.: Cardiac-specific troponin-I radioimmunoassay in the diagnosis of acute myocardial infarction. *Am. Heart J.*, 113:1333, 1987.
25. BLOOD TESTS FOR RAPID DETECTION OF HEART ATTACK © 2000 American Heart Association, Inc. Guideline, September, 2000.
26. Ebell, M.H., White, L.L., and Weismantel, D.: A systematic review of troponin T and I values as a prognostic tool for patients with chest pain. *J. Fam. Pract.*, 49:746-53, 2000.
27. Mair, J., Morandell, D., Gonser, N., et. al.: Equivalent early sensitivities of myoglobin, creatine kinase-MB mass, creatine kinase, isoform rallo, and cardiac troponins I and T for acute myocardial infarction. *Clin. Chem.*, 41: 1266-72, 1995.
28. Bertinchant, J.P., Laruc, C., Pemel, I., et. al.: Release kinetics of serum cardiac troponin I in ischemic myocardial injury. *Clin. Bio. Chem.*, 29:587-94, 1996.
29. Bodor, G.S.: Cardiac troponin I: a highly biochemical marker for myocardial infarction. *J. Clin. Immunoassay*, 17: 40-4, 1994.
30. Antman, E.M., Tanasijevic, M.J., Thompson, B., et. al.: Cardiac-specific troponin I levels to predict the risk of mortality in patients with acute coronary syndromes. *N. Engl. J. Med.*, 335:1342-9, 1996.
31. Ellestad, M.H., The diagnostic power of four chemical markers on admission to the chest pain center. In *Differential Diagnosis and Management of Patients with Chest Pain: A Multiple Biochemical Marker Approach. A Symposium Sponsored by Baylor College of Medicine, held during the 68th Scientific Session of the American Heart Association, November 11, 1995, Anaheim, California.*
32. Hanfner, S., et. al.: Cardiac troponins in serum in chronic renal failure. *Clin. Chem.*, 40:1790 1994.
33. U.S. Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030. Occupational Exposure of Bloodborne Pathogens; Final Rule. *Federal Register*; 56(235):64175, 1991.
34. USA Center for Disease Control/National Institute of Health Manual, "Biosafety in Microbiological and Biomedical Laboratories", (1984).
35. National Committee for Clinical Laboratory Standards. Protection of Laboratory Workers from Instrument Biohazards and Infectious Disease Transmitted by Blood, Body Fluids, and Tissue: Approved Guideline. NCCLS Document M29-A, 1997.
36. "Clinical Guide to Laboratory Tests". Edited by N.W. Tietz, 3rd Edition. W.B. Saunders Company, Philadelphia, PA. 19106 (1995).

TECHNICAL CONSULTATION

Call or Write: Oxis International, Inc.
 323 Vintage Park Dr.
 Foster City, CA 94404
 Tel: (650) 212-2568 Fax: (650) 212-2569

050103